

Philips MRx Cardiac Monitor/Defibrillator*	
Left Compartment	Rear Compartment
<b>Side Pocket</b>	5-Lead Cable (Attached)
<input type="checkbox"/> (1) Adult BP Cuff (Attached)	<input type="checkbox"/> V-Lead Cable (Unattached)
<input type="checkbox"/> (1) Pulse Ox Finger Sensor (Attached)	<input type="checkbox"/> (4) 5-Pack Adult Electrodes
<input type="checkbox"/> (1) Roll of Printer Paper	<input type="checkbox"/> (2) 5-Pack Pediatric Electrodes
<input type="checkbox"/> (1) Large BP Cuff	<input type="checkbox"/> (2) Razors
<input type="checkbox"/> (1) Pediatric BP Cuff	<input type="checkbox"/> (8) Electrode Skin Prep Pads [Blue]
<input type="checkbox"/> (1) Oxicliq/MRX Cable - for Adhesive Sensors	<b>Monitor</b>
<input type="checkbox"/> (1) Adult Oxicliq Sensor	<input type="checkbox"/> Printer Paper (Loaded & Functional)
<input type="checkbox"/> (1) Pediatric Oxicliq Sensor	<input type="checkbox"/> Battery A, >3 Green Dots
<b>Top Flap</b>	<input type="checkbox"/> Battery B, >3 Green Dots
<input type="checkbox"/> (1) ET CO2 Circuit - Intubated [Cylinder]	<b>AED</b>
<input type="checkbox"/> (1) ET CO2 Circuit - Non-Intubated [NC]	<input type="checkbox"/> Load Check @ 150J: _____ Joules ***
<b>Right Compartment</b>	Acceptable range is 135 to 165J, or +/-10% energy selected.
<input type="checkbox"/> (1) Defib Pad Cable	<b>Cellular Connection Device</b>
<input type="checkbox"/> (2) Multifunction Pad Pkgs - Ad/Peds ( <i>In Date</i> )	<input type="checkbox"/> Plugged In, Charged, & Turned On

- \* Philips MRx Cardiac Monitor/Defibrillators are stocked on C130, C132, and C139 (12-lead ECG capable).
- \*\* Submit Apparatus Report for any issues found. Revision Date: December 2019
- \*\*\* Record Defibrillation Joules at 150J Setting. **DO NOT** attach printed strip.

**Charlottesville-Albemarle Rescue Squad**  
828 McIntire Rd Charlottesville, VA 22902